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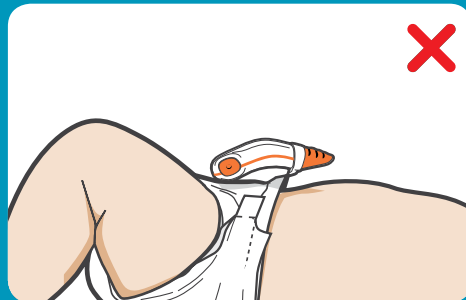
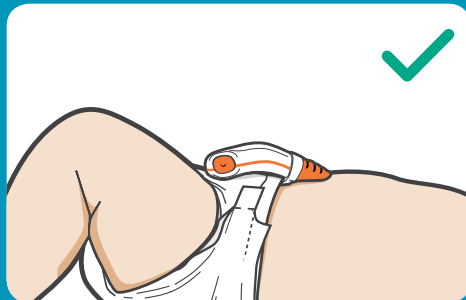
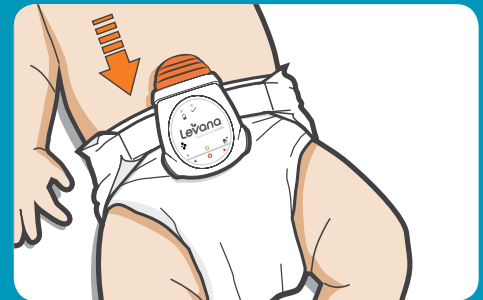
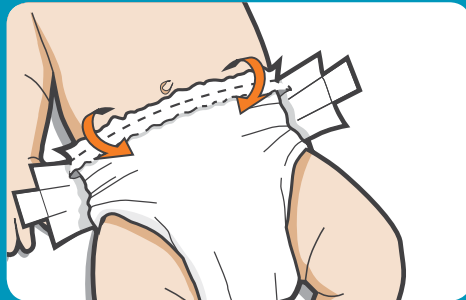
Ensure that Oma sits squarely on the diaper, and that the Movement Sensor is in contact with baby's tummy.

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## Audible Alert / Alerte Sonore / Alerta Sonora

Audible alarm sounds after 15 seconds of no abdominal movement

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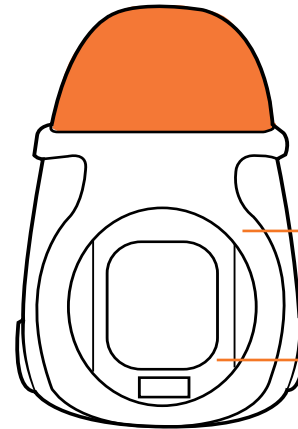
### DO NOT RETURN THIS PRODUCT TO THE STORE!

Contact Customer Support anytime for assistance.

#### Product warranty information

We take quality very seriously. This is why all of our LEVANA® products come with a one-year warranty from the original purchase date against defects in workmanship and materials. If you have warranty or support issues please contact us using any of the following methods:

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LEVANA Canada  
4080 Montrose  
Road  
Niagara Falls, ON  
Canada L2H 1J9

LEVANA USA  
60 Industrial  
Parkway, Suite #Z64  
Cheektowaga, NY  
USA 14227

- Only use monitor when baby is in a motionless bed/cot. Do not use in car seat, stroller, swing or other mobile carrier as motion negates the functionality of the monitor.
- This product is a baby care aid, not a medical device. It is not intended for the diagnosis, prevention or monitoring of a disease, injury or handicap including SIDS (Sudden Infant Death Syndrome).
- This product does not replace responsible adult supervision. Check on your child at regular intervals and ensure that the battery is full.
- The replaceable battery is designed to last for a 12 month period, being used on average 6 times per day for a total of 18 hours per day, allowing up to 4 stimulations per day and 1 alarm per day for no longer than 20 seconds.
- SNUZA is a registered trademark of Biosentronics CC.
- ©2013 LEVANA®. All rights reserved. LEVANA®, the LEVANA® logo, Oma™ and other LEVANA® marks may be registered. All other trademarks are the property of their respective owners.

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